

LEval with Jackson Strength System for Android™

User's Manual



Table of Contents

Introduction	2
Select Device Type	3
Connecting to a Device	4
Disconnecting a Device	5
Device Settings	5
Test Settings	7
Manage Organizations	11
Manage Subjects	14
View Subject Results	17
Force Over Time Chart	19
History Chart	20
Summary Chart	22
Export Database	23
Print Options	24
Contact Support	24
Icon Glossary	25
End-User License Agreement	27




Introduction

Thank you for downloading the Lafayette Evaluation (LVal) app. The LVal app allows users to connect an IOS or Android device to specific Lafayette Instrument products. One of these products is the Model 32728 Jackson Strength System. The LVal app allows Jackson users to manage test batteries, subjects, and results.

Select Device Type

Select Device Type from the Drop Down Menu

Open the navigation menu by tapping the Action Bar Drop Down Menu in the upper right corner of the screen. Select **Select Device Type** from the menu.

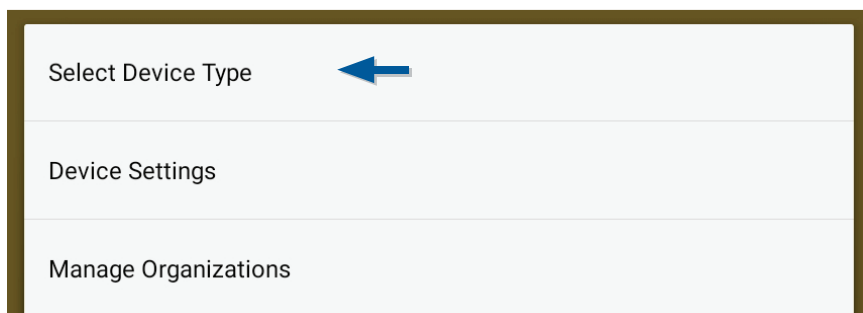

Select a Device Type

After selecting **Select Device Type**, a selection box will open allowing you to choose between available device types. Use the drop down to select which device you wish to connect. Jackson Strength System was chosen for this example.

After selecting a device type, the selected device will be loaded at the top of the screen.

Connecting to a Device

Tap Connect in Connection Bar

The Connection Bar is located directly below the Action Bar.



Select a Device to Connect

After selecting **Connect**, a new page will open allowing you to scan for available devices. All paired devices will automatically be displayed in the list. If the device you are looking for is not available in the list, tap the **Scan** button at the bottom of the dialog (see *Scan for a Device* for more information). Selecting a device that is not paired will cause a dialog box to appear asking you to pair with the device.

Scan for a Device

Scanning for a device will return a dialog box with two sets of devices: "Paired Devices" and "Other Available Devices" (other Bluetooth devices). Previously paired devices will be listed in the top section.

Pairing Code

After selecting a device, the application will attempt to connect to the device. A dialog box will be displayed asking for a Pairing Code. The Pairing Code for a Jackson Strength System is 32728.

Connection Successful


After selecting a device and pairing the device (if necessary), the application will attempt to connect to the device. Once the device has successfully connected, the Connection Bar (located below the Action Bar) will turn from red to blue, and the name of the connected device will appear. The device's current battery level is also noted by the battery icon in the Connection Bar.



Disconnecting from a Device

Tap Disconnect in Connection Bar

The Connection Bar is located directly below the Action Bar.

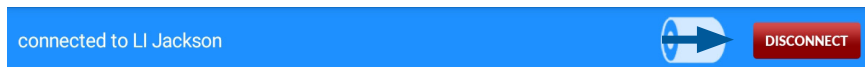

Note: If device becomes disconnected by alternative means, be sure to verify it is powered on.

Device Settings

Open the drop down menu by tapping the Action Bar Drop Down Menu in the upper right corner of the screen. Select **Device Settings** from the menu. All Device Settings are available on a single page. Once all the settings are correct, tap the **Accept Changes** button. The following explains the meaning of each setting.

A yellow rectangular screen displaying three settings. The first is "Preparation Duration: 1 second" with a slider bar below it. The second is "Test Duration: 3 seconds" with a slider bar below it. The third is "Force Units" with a dropdown menu showing "Pounds". At the bottom are two blue buttons: "Accept Changes" and "Cancel".

Preparation Duration: 1 second

Test Duration: 3 seconds

Force Units

Pounds

Accept Changes Cancel

Preparation Duration

Preparation Duration is the time between when a test is started and when the results begin recording. This section of data is ignored in order to prevent data skewing as a result of beginning to pull on the bar. From the Settings page, see **Set Preparation Duration**.

Test Duration

The test time is the total time of of the test not including any preparation duration. From the Settings page, see **Test Duration**. A slider control will allow you to select from 0 to 10 seconds.


Unit of Force

From the Settings page, see **Unit of Force**. A select box will appear allowing you to select the desired unit (pounds or kilograms) of force.

Test Settings

Select a Test

Tap the **Select a Test** icon from the Test Action Bar at the bottom of the application. A dialog box will appear with the available tests to select.

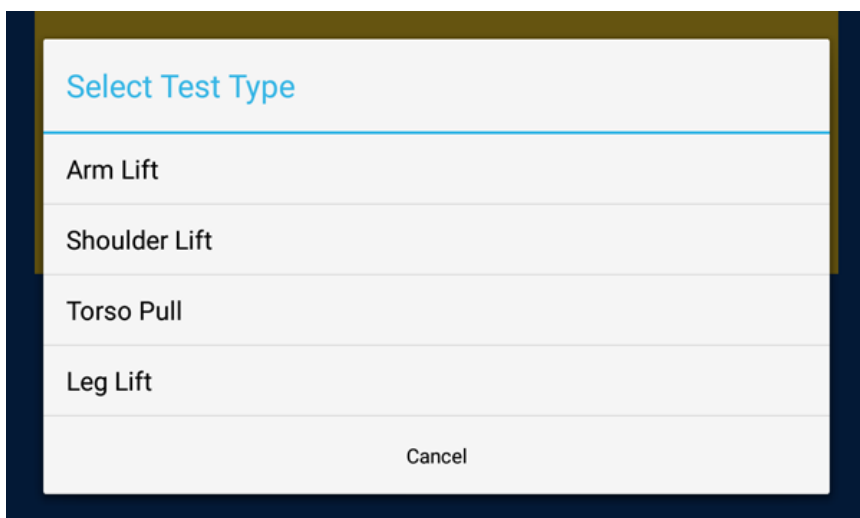

After selecting a test, the selected test and the default duration of 3 seconds will be displayed below the Connection Bar.



Get Test Zero

Tap the **Get Test Zero** icon from the Test Action Bar at the bottom of the application.



Once the test zero is calculated, a toast message will appear stating "Test Zero Calculated".

Running a Test

After getting test zero, selecting a test to run, setting the preparation duration, setting the test duration, and selecting the unit of force, a test may be ran. To start a test select the play button from the Test Action Bar at the bottom of the application.




Saving a Test

After running a test, the results may be saved. To save a test tap the **Save** icon from the Test Action Bar at the bottom of the application. A Save Test Results dialog box will appear requesting you to select the Organization and Subject where you would like to save the results. Selecting an Organization filters the available subjects.



To filter the Subject list by Organization, tap the drop down. Subject is a required field, and it can be populated by typing a name or beginning to type a name and selecting it from the populated list.

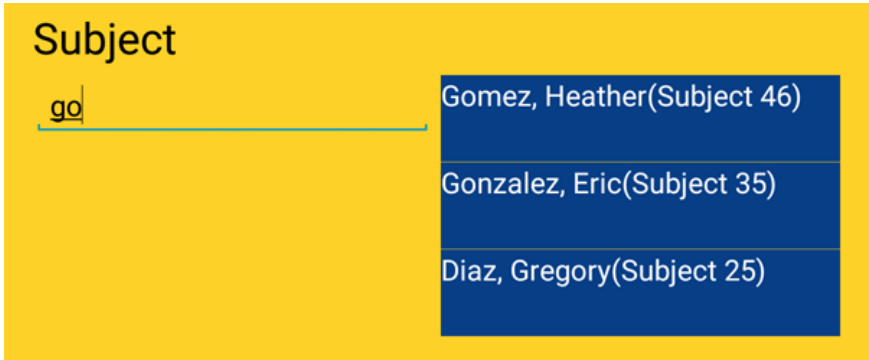A screenshot of a yellow background with the word "Subject" in bold black text. Below it is a text input field containing the text "go". To the right of the input field is a blue dropdown menu with three options: "Gomez, Heather (Subject 46)", "Gonzalez, Eric (Subject 35)", and "Diaz, Gregory (Subject 25)".

Subject
Gomez, Heather (Subject 46)
Gonzalez, Eric (Subject 35)
Diaz, Gregory (Subject 25)

Note: The Save Test form will not accept Subject names that are not already saved as subjects. The name listing that appears in blue to the right of the Subject field will scroll vertically if the name list is too long for the available space.

Once the desired Subject and Organization are selected, tap on the **Save Results** button.

Tapping the **Save Results** button will close the dialog box and a toast message will appear stating "Test Results Saved". All force values visible on the main screen will be cleared, and the unit will be ready for additional testing.

Clear Test Results

After running a test, the results can be cleared by tapping the **Clear Results** icon in the Test Action Bar at the bottom of the application.



Note: It is not necessary to clear the results before running another test.

Battery Status


After connecting to a device, the device's battery status will be displayed in the Connection Bar. The app will update the status of the battery every 60 seconds.



Manage Organizations

Manage Organizations Dialog Box

Open the navigation menu by tapping the Action Bar Drop Down Menu in the upper right corner of the screen. Select **Manage Organizations** from the menu.

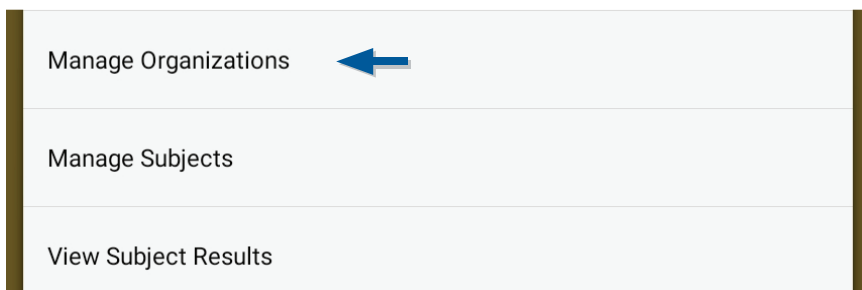

Add Organizations

After selecting **Manage Organizations** from the menu, a new page will appear with all organizations listed. Tap the **Add** icon to add a new Organization.




A new page will appear with a text area entitled "Organization Name". Type in the Organization Name then tap the **Save** icon.



Once the Organization is saved, a toast message will appear stating "Saved Organization", and the Organization list will be re-populated to include the new Organization.

Edit Organizations

After selecting **Manage Organizations** from the menu, a new page will appear with all organizations listed.

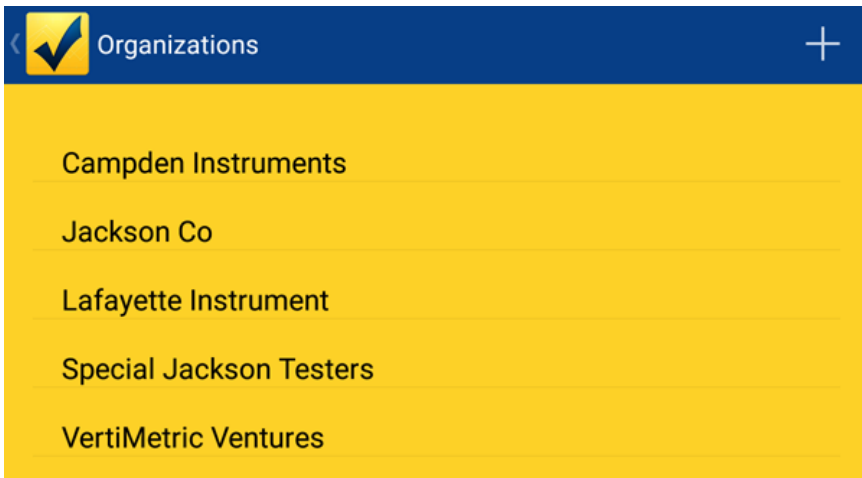

Tap the desired Organization and another page will appear. Type in the new or modified name and click the **Save** icon.




Once the Organization is saved, a toast message will appear stating "Saved Organization", and the Organization list will be re-populated to include the new Organization.

Delete Organizations

After selecting **Manage Organizations** from the menu, a new page will appear with all organizations listed. Tap the desired Organization and another dialog box will appear. Tap the **Delete** icon on the dialog box menu bar.



A dialog box will open asking you to confirm the deletion.

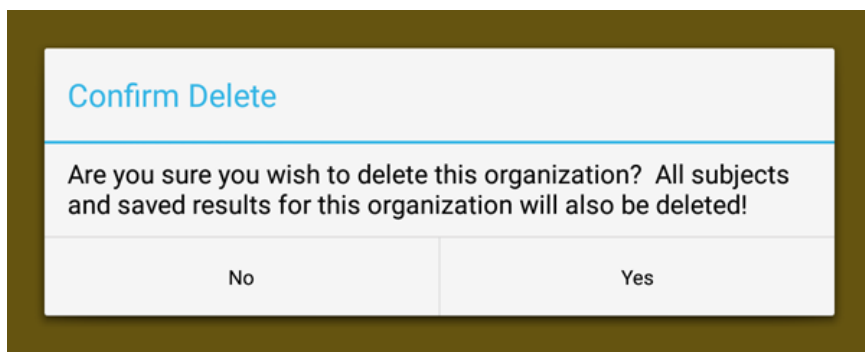


Once the deletion is confirmed, a toast message will appear stating "Organization Deleted", and the Organization list will be re-populated without the deleted Organization.

Manage Subjects

Manage Subjects Dialog Box

Open the navigation menu by tapping the Action Bar Drop Down Menu in the upper right corner of the screen. Select **Manage Subjects** from the menu. A new page will appear with all Subjects listed. Subjects may be filtered by using Organization drop down.

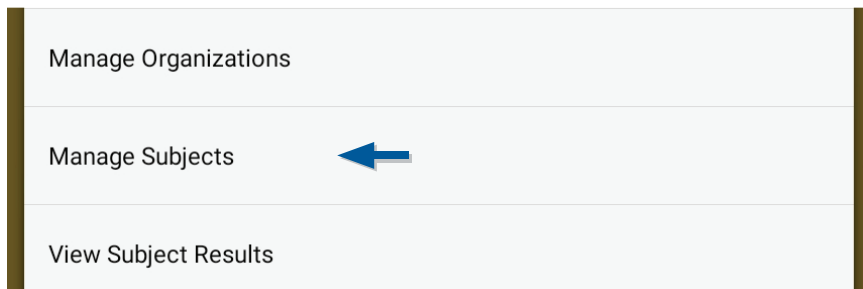

Add Subjects

After selecting **Manage Subjects** from the menu, a new page will appear with all the Subjects listed. Tap the **Add** icon to add a new Subject.



A dialog box will appear with textareas entitled "First Name", "Last Name", "More Info", and "Organization". Selecting "New Organization" from the Organization spinner will bring up the New Organization dialog box. Type in the Subject Name then tap the **Save** icon.



Once the subject is saved, a toast message will appear stating "Saved Subject", and the Subject list will be re-populated with the new Subject.

Edit Subject

After selecting **Manage Subjects** from the menu, a new page will appear with all the Subjects listed. A new page will appear with all Subjects listed. Tap the Subject and another page will appear.

Filter Subjects by Organization

All Organizations

Subjects

Bailey, Jacob

Baker, Madison

Carter, Gregory

Clark, Ralph

Cruz, Andrea


Tap the **Save** icon to complete the edit.



Once the subject is saved, a toast message will appear stating "Subject Saved", and the Subject list will be re-populated with the edited Subject.

Delete Subject

After selecting **Manage Subjects** from the menu, a new page will appear with all the Subjects listed. Tap the Subject and another dialog box will appear. Tap the **Delete** icon on the dialog box menu bar. A dialog box will open asking you to confirm the deletion.

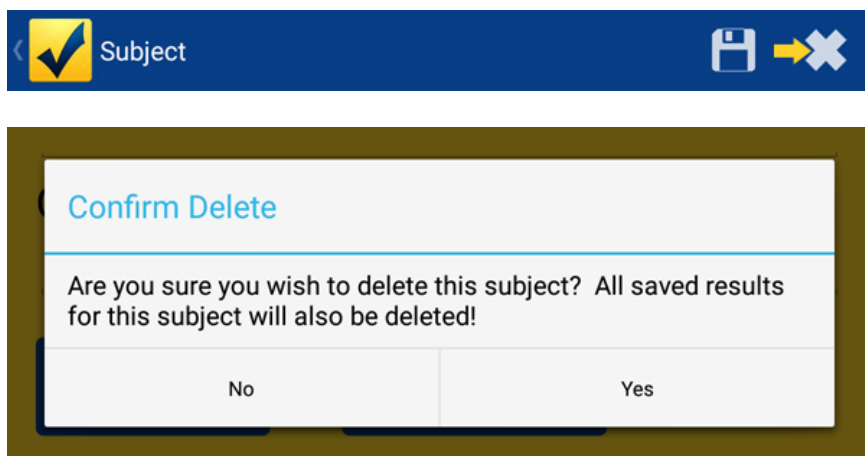

Once the deletion is confirmed, a toast message will appear stating "Subject Deleted", and the Subjects list will be re-populated without the deleted Subject.


View Subject Results

Open the navigation menu by tapping the Action Bar Drop Down Menu in the upper right corner of the screen. Select **View Subject Results** from the menu.



A new page will appear with search criteria required for the Subject Results. Submitting the form without filtering by Organization will return all results that fit between the chosen Start and End Dates.

Use the "Organization" drop down to filter the Subjects by Organization; typing in the "Subject" field populates a selection list of relevant Subject Names (filtered by the current organization).

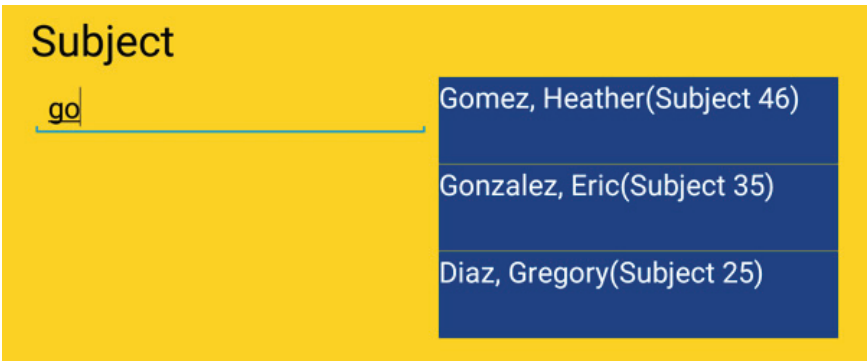

Note: The name listing that appears in blue to the right of the Subject field will scroll vertically if the name list is too long for the available space.

Tap the “Test types” drop down to open a dialog box with a list of all Test types. Select an Test by tapping on the Test’s name. Once selected, the dialog box will close, and the selected Test type will be shown in the drop down. Select a date range, and tap the **Filter Results** button.

A list displaying all results within the selected filter criteria will appear. To view details for specific results, tap the desired record in the list. Once selected, a new page will appear with additional details. To view a Force Over Time chart for the selected result, tap the **Chart** icon.

Note: See the *Force Over Time Chart* section for more details about the details of the Force Over Time chart.



From the Specific Subject Results dialog box it is also possible to navigate back to the filtered results (tap the **Back** icon) or Delete the Result (tap the **Delete** icon). Selecting to delete the result will require a Confirmation. Once deleted, the filtered results will be refreshed.



A print icon is available at the top of the result and the Force Over Time chart.



Note: See the *Print Options* section for more details about how to setup your device to print.


Force Over Time Chart

The Force Over Time chart is available from the View Subject Results area. For more information about this section, see the *View Subject Results* section.

To view the Force Over Time chart for a selected result, tap the **Chart** icon.



The Force Over Time chart plots the force exerted by elapsed time in seconds. The Subject's Name and Organization and the Test Type and Date are shown.

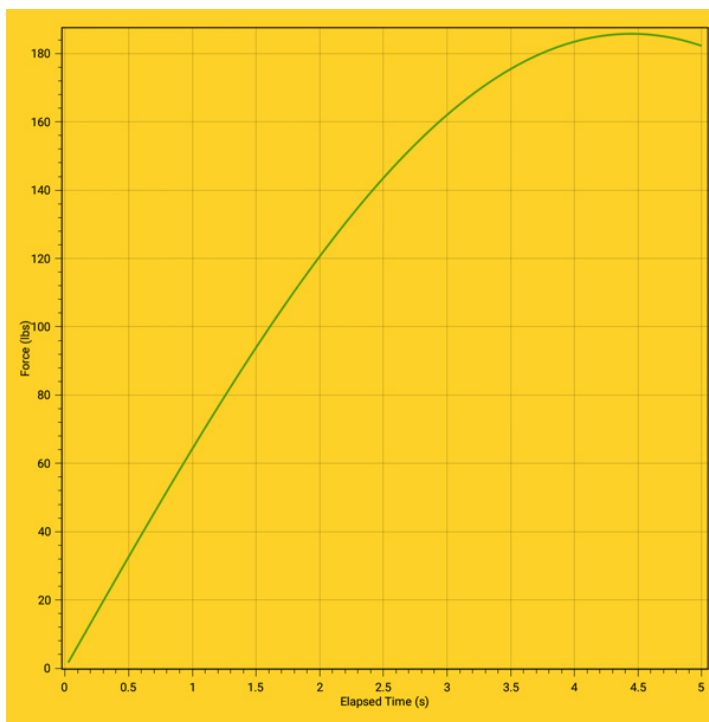

History Chart

Open the drop down menu by tapping the Action Bar Drop Down Menu in the upper right corner of the screen. Select **Manage Subjects** from the menu.


A new page will appear with a "Select organization" drop down to filter the Subjects; tap the desired subject's name to open a new page with the Subject's information. Tap on the **History Chart** button at the bottom to view the chart.

From the History Chart page, check boxes are available for each test type (Arm Lift, Shoulder Lift, Torso Pull, and Leg Lift). Checking the check box causes the corresponding data to be plotted on the chart.

☒ Arm Lift ☒ Shoulder Lift ☒ Torso Pull ☒ Leg Lift



When a single Test Type is chosen, a drop down for Average Type is available. The Average Type defaults to None.

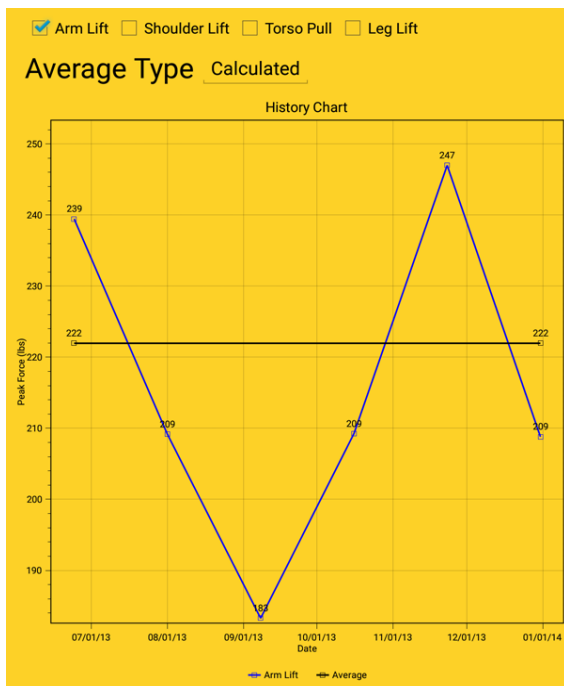

Note: Selecting Calculated causes the displayed average to be calculated by the data from the chosen Organization. Selecting Manual causes a number picker to be displayed. Either tap on the value and use the number pad or use the +/- buttons to change the value. The Average may be any value from 1-1000.

A print icon is available at the top of the chart.




Note: See the *Print Options* section for more details about how to setup your device to print.

Summary Chart

Open the drop down menu by tapping the Action Bar Drop Down Menu in the upper right corner of the screen. Select **Manage Subjects** from the menu.

A new page will appear with a “Select organization” drop down to filter the Subjects; tap the desired subject’s name to open a new page with the Subject’s information. Tap on the **Summary Chart** button to view the available Test Dates. Tap on an available Test Date to see the Chart.

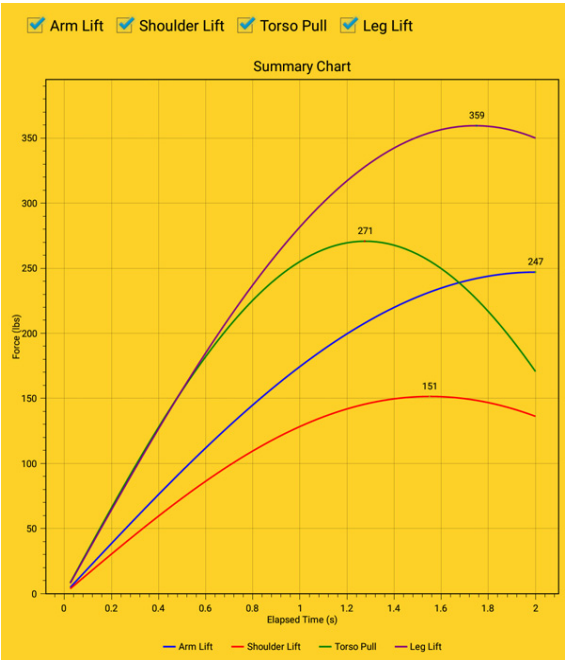

From the Summary Chart page, check boxes are available for each test type (Arm Lift, Shoulder Lift, Torso Pull, and Leg Lift). Checking the check box causes the corresponding data to be plotted on the chart.


A print icon is available at the top of the chart.

Note: See the *Print Options* section for more details about how to setup your device to print.



Export Database

Open the drop down menu by tapping the Action Bar Drop Down Menu in the upper right corner of the screen. Select **Export Database** from the menu.

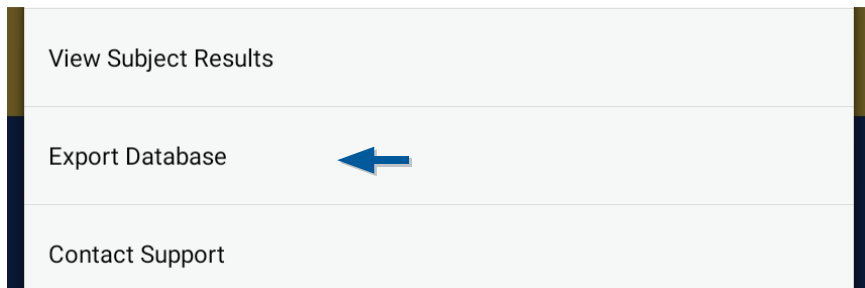

Exporting the result generates a .CSV spreadsheet and opens the mail program to send it to the email address of your choice.

Print Options

The Android platform allows printing through several functions. The print dialog allows connections for Network Printers, Cloud Printers, Printing to Google Drive, and also printing to .PDF files. For more information regarding printing to your network printer, read the printer's documentation.

Printing to Google Cloud Print registered printers:

<https://support.google.com/cloudprint/answer/1686197?rd=1>

Printing to Google Cloud Print Ready Printers:

<http://www.google.com/intl/en/cloudprint/learn/printers.html>

To print to a .PDF file, select "Save as PDF" from the print dialog box, and tap Print button. The print dialog will direct you to name the pdf and then save it in the Downloads area.

Contact Support

In the event of an unexpected error or by choosing **Contact Support** from the Action Bar drop down, the user will be prompted to submit a report to Lafayette Instrument Company.



A short form will appear asking the user to include contact information and any comments regarding the issue. The form can be filled out by tapping in each field and entering the applicable information individually. When completed, the form may be sent to our team of software engineers by tapping the **Submit** button.

Icon Glossary

The following icons can be found in the Jackson Strength System interface of the LEval app. See the section corresponding to the icon for more information.



Action Bar Dropdown: Tapping this icon brings up the main dropdown navigation for the app.



Device Battery Indicator: Non-tappable icon that shows the current status of the connected device's battery.



Test Type: Tapping this icon brings up a listing of available test types for the Jackson Strength System.



Set Device Zero: Tapping this icon sets zero measurement for the connected Jackson Strength System.



Run Test: Tapping this icon begins the selected test type with the chosen parameters.



Save Test Results: Tapping this icon brings up a form to select a subject and save the previous test results.



Clear Test Results: Tapping this icon clears previous test's results without saving.



Add: Tapping this icon brings up a form to add a new item.



Save: Tapping this icon saves the current item.



Delete: Tapping this icon deletes the current item. Deleting a subject removes all test results for that subject. Deleting an organization removes all subjects and test results for that organization.



Force-Over-Time Chart: Tapping this icon brings up a Force-Over-Time Chart for the chosen test results.



Print: Tapping this icon brings up a print dialog box.

End-User License Agreement

=====

Lafayette Instrument Company - SOFTWARE LICENCE AGREEMENT

=====

READ THE TERMS OF THIS AGREEMENT AND ANY PROVIDED SUPPLEMENTAL LICENSE TERMS (COLLECTIVELY "AGREEMENT") CAREFULLY BEFORE CONTINUING WITH THE INSTALLATION OF THIS SOFTWARE. BY CONTINUING TO INSTALL THIS SOFTWARE, YOU INDICATE YOUR AGREEMENT TO THE TERMS OF THIS LICENCE. IF YOU DO NOT AGREE TO ALL THESE TERMS, PROMPTLY CEASE INSTALLATION OF THE SOFTWARE.

- 1. LICENSE TO USE** - Lafayette Instrument grants you a non-exclusive and non-transferable license for the internal use only of the accompanying software and documentation and any error corrections provided by Lafayette Instrument (collectively "Software"), by the number of users and the class of computer hardware for which the corresponding fee has been paid.
- 2. RESTRICTIONS** - The Software is confidential and copyrighted. Title to the Software and all associated intellectual property rights is retained by Lafayette Instrument and/or its licensors. Except as specifically authorized in any Supplemental License Terms, you may not make copies of the Software, other than a single copy of the Software for archival purposes. Unless enforcement is prohibited by applicable law, you may not modify, decompile, or reverse engineer the software. No right, title or interest in or to any trademark, service mark, logo or trade name of Lafayette Instrument or its licensors is granted under this Agreement.
- 3. LIMITED WARRANTY** - Lafayette Instrument warrants to you that for a period of ninety (90) days from the date of purchase, as evidenced by a copy of the receipt, the media on which this

Software is furnished (if any) will be free of defects in materials and workmanship under normal use. Except for the foregoing, the Software is provided "AS IS". Your exclusive remedy and Lafayette Instrument's entire liability under this limited warranty will be at Lafayette Instrument's option to replace the Software media or refund the fee paid for the Software.

- 4. DISCLAIMER OF WARRANTY** - Unless specified in this agreement, all express or implied conditions, representations and warranties, including any implied warranty of merchantability, fitness for a particular purpose or non-infringement are disclaimed, except to the extent that these disclaimers are held to be legally invalid.
- 5. LIMITATION OF LIABILITY** - To the extent not prohibited by law, in no event will Lafayette Instrument or its licensors be liable for any lost revenue, profit or data, or for special, indirect, consequential, incidental or punitive damages, however caused regardless of the theory of liability, arising out of or related to the use of or inability to use the software, even if Lafayette Instrument has been advised of the possibility of such damages. In no event will Lafayette Instrument's liability to you, whether in contract, tort (including negligence), or otherwise, exceed the amount paid by you for Software under this Agreement. The foregoing limitations will apply even if the above stated warranty fails of its essential purpose.
- 6. TERMINATION** - This Agreement is effective until terminated. You may terminate this Agreement at any time by destroying all copies of the Software. This Agreement will terminate immediately without notice from Lafayette Instrument if you fail to comply with any provision of this Agreement. Upon Termination, you must destroy all copies of Software.

- 7. EXPORT REGULATIONS** - All Software and technical data delivered under this Agreement are subject to United States of America (USA) export control laws and may be subject to export or import regulations in other countries. You agree to comply strictly with all such laws and regulations and acknowledge that you have the responsibility to obtain such licenses to export, re-export, or import as may be required after delivery to you.
- 8. GOVERNING LAW** - This Agreement is governed by and shall be construed in accordance with the laws of The United States Of America (USA) and you submit to the exclusive jurisdiction of the USA courts.
- 9. SEVERABILITY** - If any provision of this Agreement is held to be unenforceable, this Agreement will remain in effect with the provision omitted, unless omission would frustrate the intent of the parties, in which case this Agreement will immediately terminate.
- 10. INTEGRATION** - This Agreement is the entire agreement between you and Lafayette Instrument relating to its subject matter. It supersedes all prior or contemporaneous oral or written communications, proposals, representations and warranties and prevails over any conflicting or additional terms of any quote, order, acknowledgment, or other communication between the parties relating to its subject matter during the term of this Agreement. No modification of this Agreement will be binding, unless in writing and signed by an authorized representative of each party.



3700 Sagamore Pkwy N

Lafayette, IN 47904 USA

Tel: (765) 423-1505 • (800) 428-7545

Fax: (765) 423-4111

info@lafayetteinstrument.com

www.lafayetteinstrument.com